
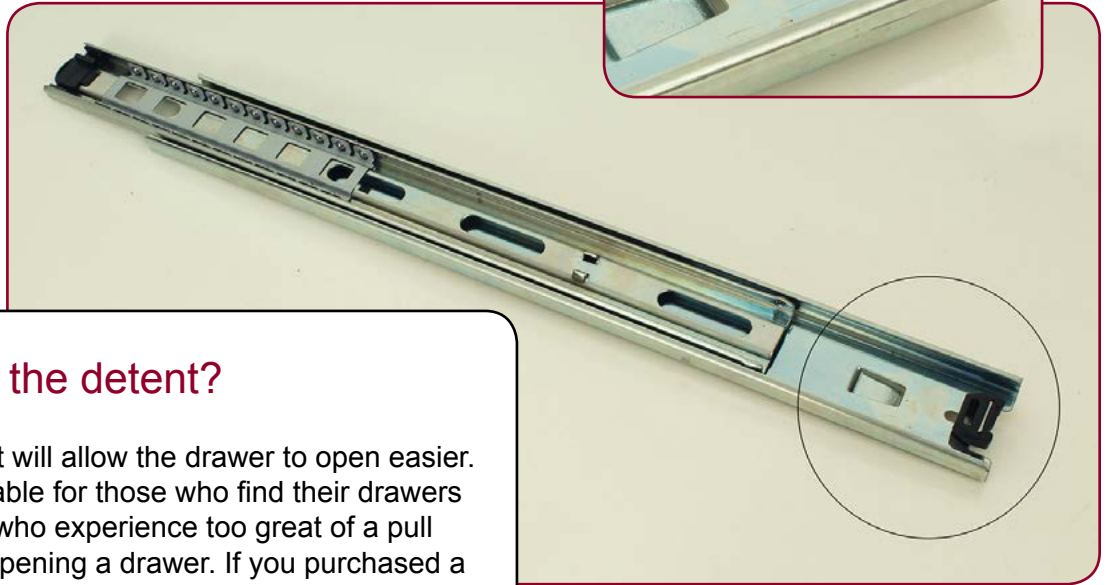


What is a detent?

The detent is a small plastic piece on the back of each drawer slide that keeps the drawer from coming open when carts are shaken or moved. On smaller Harloff carts, you really only need a detent on one side of each drawer to prevent the unwanted opening of drawers.

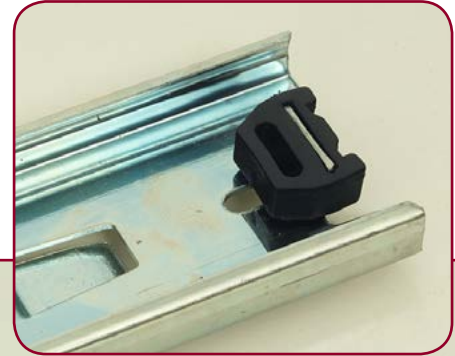

Why remove the detent?

Removing a detent will allow the drawer to open easier. This may be desirable for those who find their drawers are too “sticky” or who experience too great of a pull on the cart when opening a drawer. If you purchased a cart without locking casters, you may wish to remove detents from one side of your cart in order to prevent the cart from moving with each drawer pull.

How do you remove a detent?

Completely remove the cart drawer which you want to modify from its drawer slides. The detent is located at the back of each drawer slide, on a small metal holder. Simply pull the small, black plastic detent straight out from its holder. We recommend that you remove a detent from only one side of a drawer for optimal function. Re-insert the drawer and you're set!

