


SCW Scope Mount Replacement

Instructions:

1. The setup of your current scope holder looks something like this ▷
2. There are three screws in the marked locations. Holding the mounted scope holder in one hand and a large, Phillips head screwdriver in the other hand, remove each of these screws and then remove the holder.
3. In the area where the scope holder was removed, you will see three vertical rows of nutserts: two on the left, two in the middle and two on the right. This is where you will mount the new scope holder.

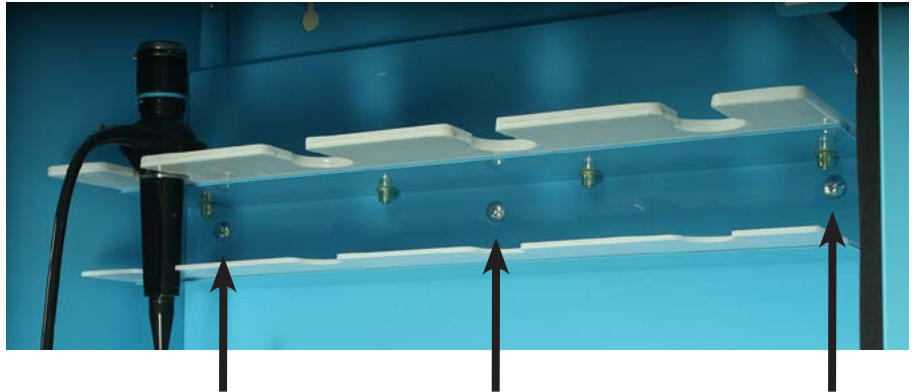

4. Your new five scope holder has three mounting holes on each side. You will use only two of these to screw into your SCW cabinet, depending on whether you want the holder slightly lower or slightly higher in the cabinet.



5. Your scope holder came with four mounting screws. Use your screwdriver to mount the new scope holder into the left two nutserts and right two nutserts in your cabinet. Hand tighten.
6. All done! Your new scope holder should look this this ▷

