


# HEAT SEALED POWDER COATED JET CART SCOPE DRYING SYSTEM

STC-AL6-JET (6-scope capacity)

STC-AL8-JET (8-scope capacity)

DATE ISSUED:  
September 2025

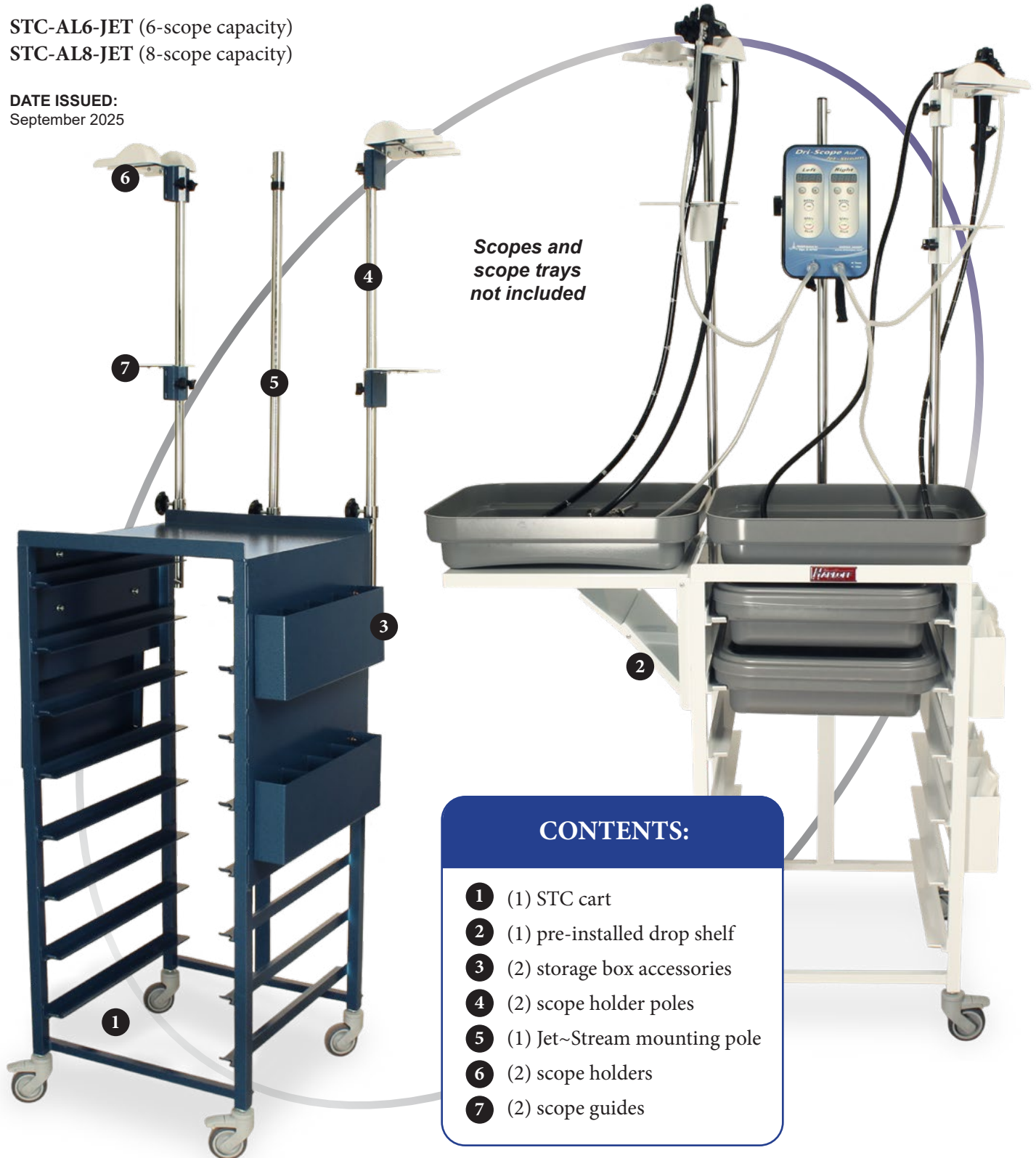

# Jet Cart Initial Setup

Jet Carts come pre-installed with a drop shelf, accessory mounting screws, scope holders/guides and mounting poles set in the lowest position.

## Preparation for Cart Use:

All mounting poles, scope holders and scope guides are adjustable and use compression screws to keep them in place. To ready your Jet Cart for use, do the following:



1. Choose any mounting pole and unscrew (counter-clockwise) the adjustment handle until the pole is able to slide up or down. Extend the mounting pole to its highest position, but do not pull the pole higher than the adjustment screw holder. Rotate the adjustment screw clockwise to tighten it, locking the pole back into position. ▷
2. Repeat this for each mounting pole so that all 3 poles are extended.
3. On the back of each scope holder are two adjustment screws. Unscrew (counter-clockwise) both screws on the adjustment handle on one of the scope holders until it is able to slide up or down. Slide the scope holder upward until it is near the top of the mounting pole. Rotate the adjustment screws clockwise to tighten them so they lock the into position. Repeat this process for both scope holders. ▷
4. The scope guides use the same type of adjustment screws as the scope holders. Using the same process as in step 3, mount them anywhere in the middle of the mounting pole depending on your preference. Guides separate scopes from the umbilical and other surfaces, helping to keep scopes sterile while drying.
5. Jet~Stream units mount on the middle pole at your desired height. They use similar compression screws as the mounting pole for adjustment. Please consult your Dri-Scope Aid® documentation **402119-214** for details about mounting and unit operation. ▷



# Accessories & Scope Drying Setup

## Accessories:

1. To operate the drop shelf, pull open by lifting the shelf up. It will automatically lock into an extended position. To retract the shelf, slightly lift it with one hand and push the “elbow” area under the shelf with your other hand.

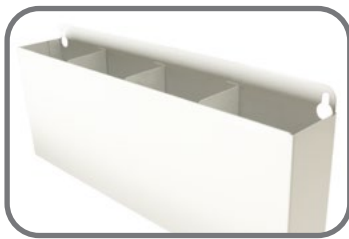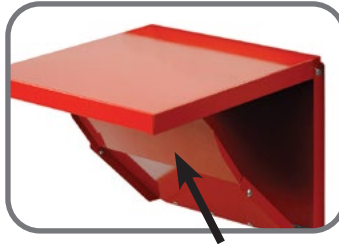

2. To mount your storage boxes, simply hang the left and right side eyelets on the mounting screws of the non-shelf side of your STC cart.

## Scope Drying Setup:

1. Endoscopes mount on scope holders with the valves at the front with the insertion tube hanging down through the “forks” of the holder. The umbilical cord/connector should hang down the back. Scope holders can be mounted on the poles facing any direction that works best for your setup.



2. The image at the top right shows a typical drying setup. The inset images at the bottom right show alternative setups where scopes remain in the tray. Select a setup that meets your guidelines.

3. Please consult your Jet~Stream documentation for setup of tubing and for unit operation.

